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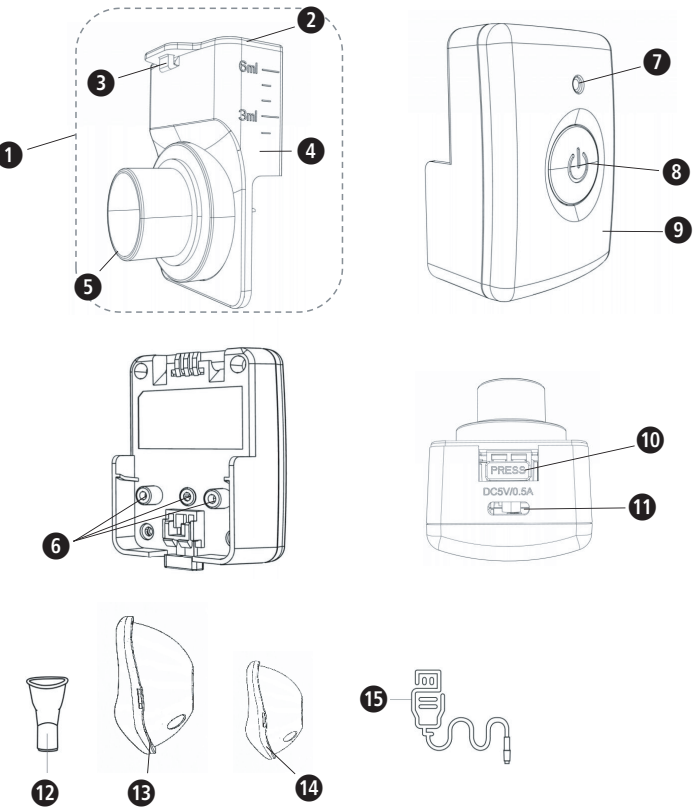
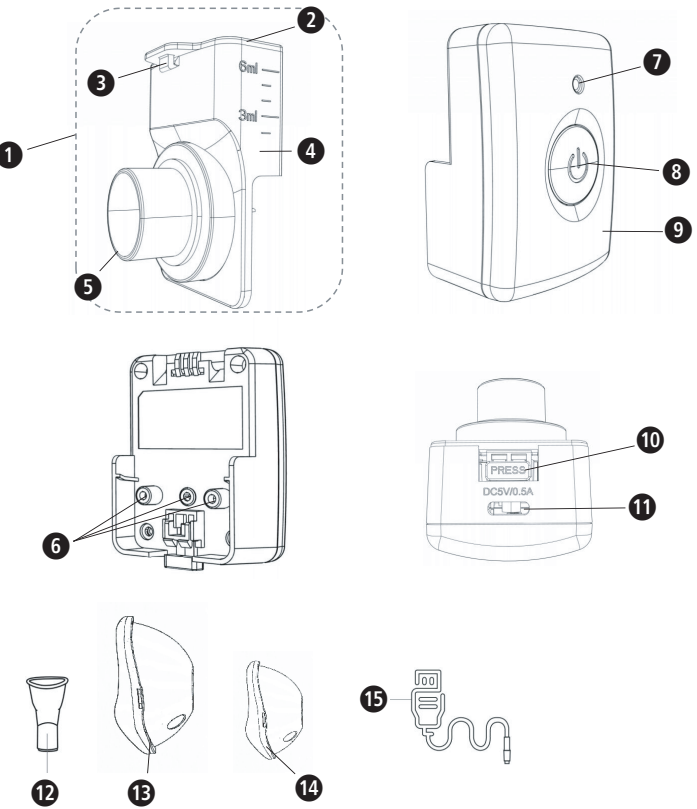
Device and controls

- 1 Nebulizer head 2 Cover 3 Lock
- 4 Medicine cup 5 Aerosol head 6 Electrodes
- 7 Power indicator 8 ON/OFF button 9 Main unit
- 10 Disassembly button 11 Charging cable port 12 Mouthpiece
- 13 Adult mask 14 Child mask 15 USB-C cable

Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING
These warning notes must be observed to prevent any injury to the user.

CAUTION
These notes must be observed to prevent any damage to the device.

NOTE
These notes give you useful additional information on the installation or operation.

Device classification: type BF

Authorized EU representative

LOT	LOT number	SN	Serial number
MD	Medical device		
	Ambient pressure limitation		Humidity range


Recycling symbols/codes:
These are used to provide information about the material and its proper use and recycling.

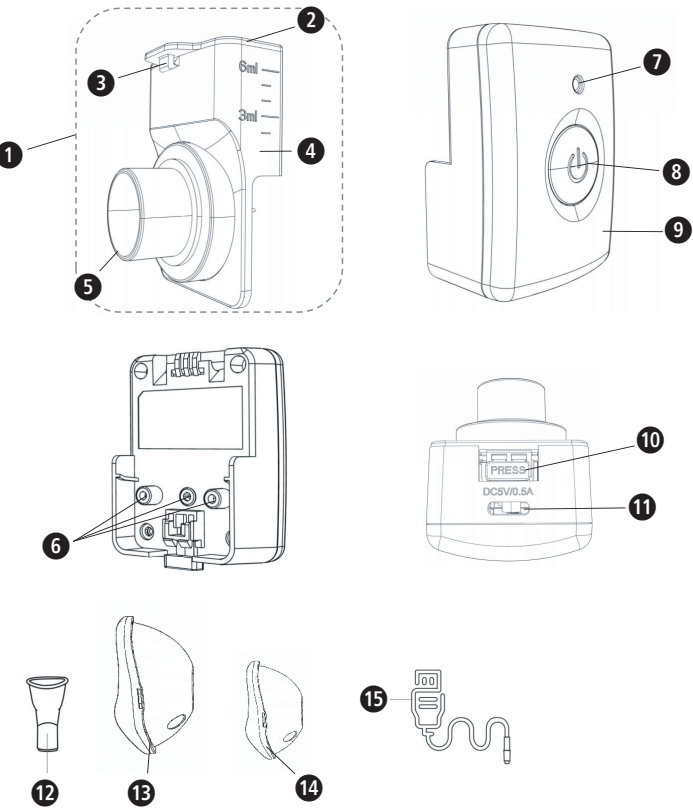IP25
Protected against solid foreign objects 12.5 mm in diameter and larger, and directly affected by water spray in any direction without harmful effects.

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