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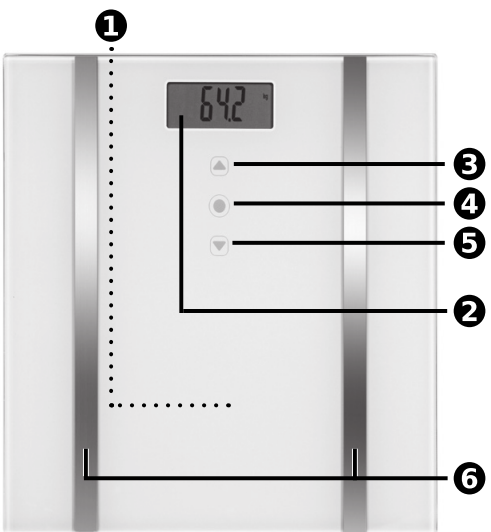
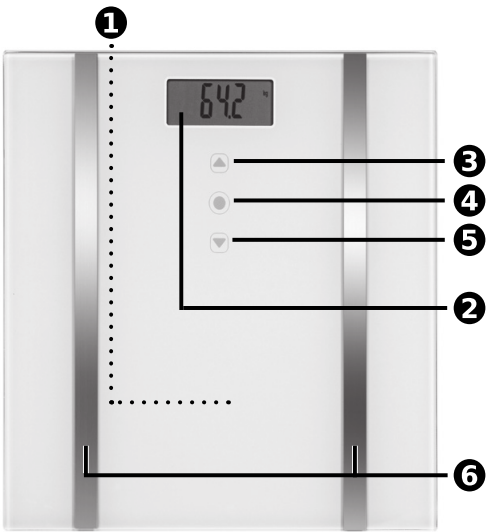
GB *Instruction Manual*
Body Analysis Scale BS 483

Device and controls

- ❶ Battery compartment (on underside)
- ❷ Display
- ❸ ▲ Up button
- ❹ SET button
- ❺ ▼ Down button
- ❻ Electrodes

Items supplied and packaging

Please check first of all that the device is complete and is not damaged in any way.

In case of doubt, do not use it and contact your supplier or your service centre.

The following parts are included:

- 1 Body Analysis Scale **BS 483**
- 3 Batteries (Type AAA, 1.5 V)
- 1 Short instructions

♻️ The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your supplier without delay.

⚠️ **WARNING**
Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

Explanation of symbols

🏠 Only use indoors!

📖 **IMPORTANT!**
Non-observance of these instructions can result in serious injury or damage to the device.

⚠️ **WARNING**
These warning notes must be observed to prevent any injury to the user.

⚠️ **CAUTION**
These notes must be observed to prevent any damage to the device.

ℹ️ **NOTE**
These notes provide useful additional information on the installation or operation.

🛡️ Protection category II

🏢 **LOT** LOT number

🏭 Manufacturer

♻️ Recycling symbols/codes:
These are used to provide information about the material and its proper use and recycling.

GB **Safety instructions**

📖 **Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.**

- ⚠️ ⚠️ ⓘ
- These scales are not suitable for people with pacemakers or other medical implants. The body fat information may be inaccurate in people with diabetes or other medical/physical limitations. The same also applies to very highly trained athletes.
- This device is not suitable for pregnant women!
- The scale has been manufactured for domestic use. It is not suitable for commercial use in hospitals or other medical institutions.
- Do not use the scale in a location where extremely high temperatures or humidity may occur. Keep the scale away from water.
- Do not place the scale on an uneven surface. Do not stand on the corners/edges of the scale. When weighing yourself, do not stand on one side or the edge of the scales.
- Danger of slipping! Do not stand on the scale with wet feet. Do not stand on the scale wearing socks.
- The scales have a measuring range up to 180 kg (396 lb / 28 st 4 lb). Do not overload the scales – this can cause irreversible damage.
- Handle the scale with care. The scale must not be subjected to impacts or vibration. Do not fall or drop anything on the scales.
- Do not attempt to repair the device yourself in the event of a malfunction since this will invalidate the warranty. Repairs should only be carried out by authorised service centres.
- Children shall not play with the appliance.
- In the event of malfunctions, do not proceed with any unauthorised repair of the device. Otherwise, this will render any warranty claims void. Have repairs performed only by an authorised service centre.

⚠️ **WARNING – BATTERY SAFETY INFORMATION**

- **Do not disassemble batteries!**
- **Clean the battery and device contacts if necessary before putting in the batteries!**
- **Remove discharged batteries from the device immediately!**
- **Increased risk of leakage, avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of this parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!**
- **If a battery has been swallowed seek medical attention immediately!**
- **Replace all of the batteries simultaneously!**
- **Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!**
- **Insert the batteries correctly, observing the polarity!**
- **Keep the battery compartment well sealed!**
- **Remove the batteries from the device if it is not going to be used for an extended period!**
- **Keep batteries out of children's reach!**
- **Do not attempt to recharge these batteries! There is a danger of explosion!**
- **Do not short circuit! There is a danger of explosion!**
- **Do not throw into a fire! There is a danger of explosion!**
- **Keep unused batteries in their packaging away from metal objects in order to prevent short circuiting!**
- **Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!**

Insert / replace battery

Before you start up your digital scale, insert the three provided 1,5 V batteries (type AAA) into the device. To do this, open the battery compartment ❶ beneath the device and insert the batteries. Make sure the polarity is correct (as shown in the battery compartment). Put the lid of the battery compartment back on and push it until it clicks into place. Change the batteries when the battery change symbol „Lo“ appears in the display or when nothing is shown in the display after the device is switched on.

Change the unit of weight

You may have your weight displayed in kilograms (kg), pounds (lb) or stones (st). Press your foot on the middle of the surface of the scales. Then press ❸ or ❺ to select the unit.

Weighing, “step-on” function

- Make sure that the scale is on firm and level ground. If you want to weigh yourself, but have moved the scale immediately before, you must first initialise the scale. Briefly press the middle of the scale surface with your foot. The display will show “0.0”. When the scale has turned off, it is ready for the “step-on” function. If the scale has not been moved beforehand, this process is omitted.
- Step onto the scales and stand still. The device switches on automatically and the display ❷ briefly shows “0.0 kg”.
- Your weight is measured, flashes twice and is then finally displayed.
- Step down from the scales. The scales switch off automatically after 15 seconds.

Programming personal data

The scales can store data for up to 10 people: sex, age, height. The scales have preset values. These are: kg, female, 25 years old, 165 cm.

- Press your foot on the middle of the surface of the scales. “0.0 kg” appears on the display.
- Press the SET ❹ button. The memory location is flashing in the display. Now press ▲ Up button ❸ or ▼ Down button ❺ to select the desired memory location (P0 - P9) for the user profile.
- Press the SET-button ❹ to confirm the selection. Now the symbol for the sex is flashing in the display.

👤 = Man | 👤 = Woman

- Set your sex by pressing the ▲ Up button ❸ or ▼ Down button ❺ to select the appropriate symbol in the display.
- Press the SET 4 button to confirm the selection. The preset height starts flashing in the display.
- Select your height using the ▲ Up button ❸ or ▼ Down button ❺.
- Press the SET 4 button to confirm the selection. The preset age starts flashing in the display.
- Select your age by pressing the ▲ Up button ❸ or ▼ Down button ❺.
- Press the SET 4 button to confirm the selection.
- The settings are now concluded.

Weighing and measuring the body analysis values

- Press your foot on the middle of the surface of the scales. “0.0 kg” appears on the display.
- Press the SET ❹ button. The last used personal data is flashing in the display.
- Now press button ❸ or button ❺ to select the desired memory location (P0 - P9) for the user profile.
- Your personal data is displayed.
- Wait until “0.0” appears on the display. Step barefoot onto the scales and stand still. Place your feet on the electrodes ❻. First, your weight is displayed.
- The values measured for body fat [fat], body water [TBW], muscle mass [💪], bones mass value [🦴], KCAL and BMI are shown one after another in the display. The series of measured values is repeated twice in the display. The scales then switch off automatically. Step off the scales.

NOTE: The data in the following table is for orientation purposes only. If you have any questions about your proportion of body fat, please consult your doctor.

Body fat proportion range in %

Age	female (%)				male (%)			
	low	normal	high	very high	low	normal	high	very high
6–39	< 21	21–33	33–39	>39	< 8	8–19	19–25	>25
40–59	< 23	23–35	35–40	>40	< 11	11–22	22–28	>28
60–100	< 24	24–36	36–42	>42	< 13	13–25	25–30	>30

Error messages

- **Err** = The scale is overloaded.
- **Lo** = The battery is low and must be replaced.
- **Err2** = The body fat and body water values cannot be determined.

If the scales do not function to your satisfaction, check the following before contacting a service centre:

- Check whether the battery has been inserted correctly.
- Check whether the correct unit of weight has been selected.
- Check whether the scales are freestanding on a solid and even surface. They must not be touching a wall or any other object.
- Weigh yourself again.

Care and maintenance

Never use aggressive cleaning agents or stiff brushes. Clean the scale using a soft and lightly moistened cloth. Never use abrasive cleaning agents or alcohol. Do not allow any water to get into the unit. Do not use the unit again until it is completely dry.

🗑️ **Disposal:** This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Please remove the battery before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Batteries containing pollutant substances are marked as follows: Cd = contains cadmium, Hg = contains mercury, Pb = contains lead.

Technical specifications

Name and model:	medisana Body Analysis Scale BS 483
Voltage supply	4.5 V (3 × 1.5V, Type AAA)
Display	Digital display
Memory	for 10 persons
Measuring range	up to 180 kg or 396 lbs or 28 st 4 lb
Maximum measuring deviation	± 1 %
Body fat	4 - 85 % in steps of 0,1 %
Body water	20 - 85 % in steps of 0,1 %
Muscle mass	13 - 80 % in steps of 0,1 %
Bone mass	0,5 - 8 kg in steps of 0,1 kg
Graduation	100 g / 0.2 lb
Autom. switch-off	after approx. 15 seconds
Dimensions (L x W x H)	30 x 30 x 2 cm / Weight approx.: 1,55 kg
Article number:	40449
EAN No.:	4015588 40449 8

Warranty and repair terms

Your statutory warranty rights are not restricted by our guarantee below. Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

1. The warranty period for **medisana** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

🏢 medisana GmbH, Carl-Schurz-Str. 2, 41460 NEUSS, GERMANY

The service centre address is shown on the attached leaflet.

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

The current version of this instruction manual can be found under www.medisana.com

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