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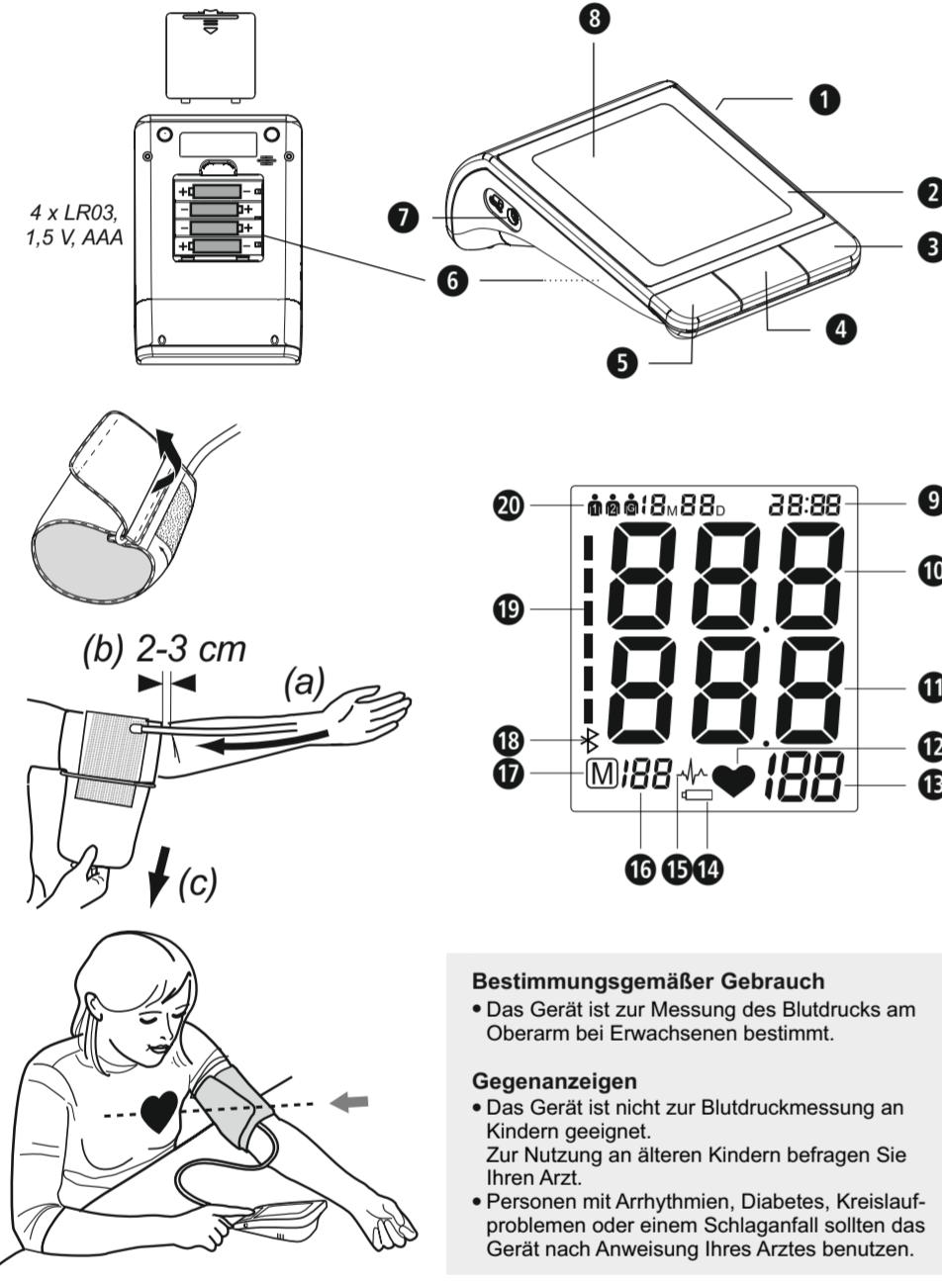
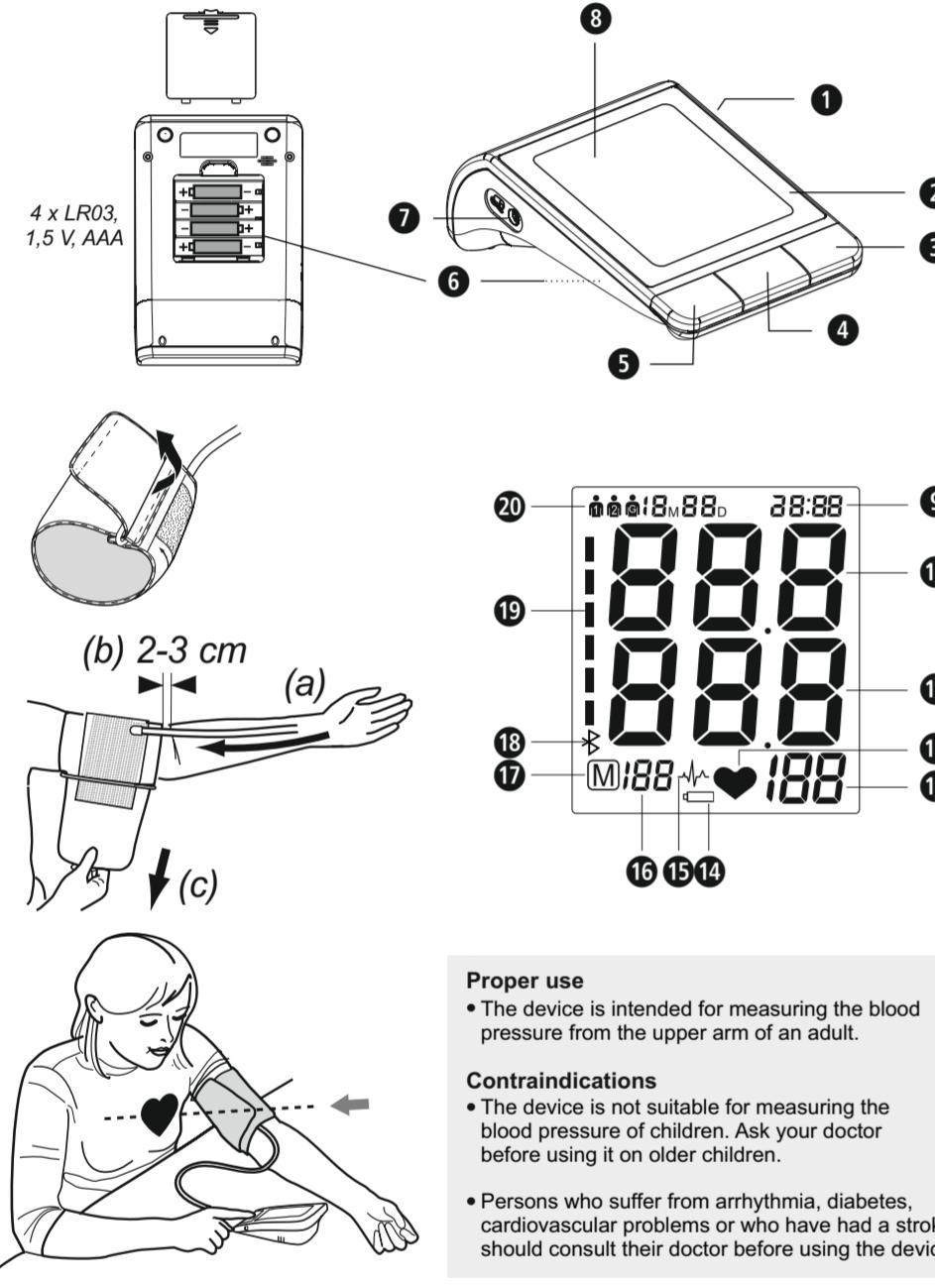
- Sie können

Blood Pressure Monitor BU 530

Instruction Manual
Please read carefully!

GB Unit and LC-Display

Explanation of symbols

IMPORTANT
Follow the instructions for use!
Non-observance of these instructions can result in serious injury or damage to the device.

WARNING
These warning notes must be observed to prevent any injury to the user.

CAUTION
These notes must be observed to prevent any damage to the device.

NOTE
These notes give you useful additional information on the installation or operation.

Device classification: type B applied part

LOT

LOT number

Manufacturer

Date of manufacture

CE 0297

GB Safety Instructions

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

Blood pressure classification WHO

These values have been specified by the World Health Organisation (WHO) without making any allowance for age.

| | | |
|-----------------------|--------------------|-------------------|
| Low blood pressure | systolic <100 | diastolic <60 |
| Normal blood pressure | systolic 100 - 139 | diastolic 60 - 89 |

Forms of high blood pressure / hypertension

| | | |
|-----------------------|-------------------------|----------------------------------------|
| Mild hypertension | (yellow display area ⑩) | systolic 140 - 159 diastolic 90 - 99 |
| Moderate hypertension | (orange display area ⑪) | systolic 160 - 179 diastolic 100 - 109 |

| | | |
|---------------------|----------------------|--------------------------------|
| Severe hypertension | (red display area ⑫) | systolic ≥ 180 diastolic ≥ 110 |
|---------------------|----------------------|--------------------------------|

- The unit is intended only for use in the home. Consult your doctor prior to using the blood pressure monitor if you are concerned about health matters.
- Only use the unit for its intended purpose in accordance with the instructions for use. Warranty claims become void if the unit is misused.
- Please consult your doctor prior to using the unit if you suffer from illnesses such as arterial occlusive disease.
- The unit may not be used to check the heart rate of a pacemaker.

- Pregnant women should take the necessary precautions and pay attention to their individual circumstances. Please consult your doctor if necessary.
- Consult your physician to get more information about your blood pressure. A self-diagnosis and self-treatment based on the measured values may be dangerous. Always follow the instructions given by your physician!

- If you experience discomfort during a measurement, such as pain in the upper arm or other complaints, press the START/STOP-button ④ to release the air immediately from the cuff. Loosen the cuff and remove it from your arm.
- The device only works correctly with the fitting cuff.
- The unit is not suitable for children.

- Measure your blood pressure before meals.

- Before taking readings, allow yourself at least 5-10 minutes rest.

- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

- WARNING**
Blood pressure that is too low represents just as great a health risk as blood pressure that is too high! Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic)!

Influencing and evaluating readings

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least 5-10 minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

Starting up

Insert / replace batteries

You must insert the batteries provided before you can use your unit. The lid of the battery compartment ⑥ is located on the underside of the unit. Open it by slightly pushing it and moving it upwards. Remove it and insert the 4 AAA LR03 type 1.5V batteries supplied. Ensure correct polarity when inserting the batteries (as marked inside the battery compartment). Close the battery compartment. Replace the batteries if the change battery symbol ⑭ appears on the display ⑩ or if nothing appears on the display after the unit has been switched on.

Using a Mains Adaptor

Alternatively, you can use the unit with a special mains adaptor (MEDISANA Art. No. 51125). This connects to the socket ① provided at the back of the unit. In this case the batteries remain in the unit. Inserting the connector into the back of the blood pressure monitor switches the batteries off mechanically. It is necessary, therefore, to plug the mains adaptor into the mains socket first of all and then to connect it to the blood pressure monitor. If the blood pressure monitor is no longer being used, first remove the connector from the blood pressure monitor and then pull the mains adaptor out of the mains socket. Doing this means that the date and time will not have to be reset each time.

Settings

1. User setting:

When unit is switched off, press and hold SET-button ⑤. ①, ② or ③ appear on the display. By pressing MEM-button ③ you may choose between user ①, ② or ③. Press SET button ⑤ to confirm the user setting. Afterwards, you can proceed with the year setting mode.

Hint: If you choose ③, measurement results will not be stored.

2. Year setting:

The year figure starts to flash for setting. Keep pressing the MEM button ③ until the actual year figure appears. Press SET button ⑤ to confirm the year setting. Afterwards, you can proceed with the month/date setting mode.

3. Month / date setting:

The month figure starts to flash for setting. Keep pressing the MEM button ③ until the actual month figure appears. Press SET button ⑤ to confirm the month setting. Afterwards, the day needs to be set in the same way. Afterwards, you can proceed with the time setting mode.

4. Time setting:

The hour figure starts to flash for setting. Keep pressing the MEM button ③ until the actual hour figure appears. Press SET button ⑤ to confirm the hour setting and proceed in the same way for minute setting. Conclusively, CL appears in the display. You may delete stored values at this point - please check chapter "Memory - clear of measurements". Press SET button ⑤ to leave the setting mode. If the batteries are changed, the set information is lost and needs to be input again.

Fitting the cuff

- Push the end piece of the air tube ⑦ into the hole on the left side of the unit prior to use.
- Slide the open end of the cuff through the metal bracket so that the Velcro fastener is on the outside and it becomes a cylindrical form (Fig.1). Slide the cuff over your left upper arm.

- Position the air hose in the middle of your arm in line with your middle finger (Fig.2) (a). The lower edge of the cuff should be 2 - 3 cm above the crease of the elbow. (b). Pull the cuff tight and close the Velcro fastener (c).

- Measure the pulse on your bare arm.
- Only position the cuff on the right arm if it cannot be used on the left arm. Always carry out measurements on the same arm.

- Correct measuring position for sitting (Fig.3).

Taking a blood pressure measurement

After the cuff has been appropriately positioned, the measurement can begin:
1. Switch the unit on by pressing the START/STOP button ④.

2. If the START/STOP button ④ is pressed, all display characters are shown (display test). This test can be used to check that the display is indicating properly and in full.

3. The unit is ready for measurement. The number 0 flashes for 2 seconds and two short beeps are emitted. The device then automatically inflates the cuff slowly in order to measure your blood pressure. The rising pressure in the cuff is shown on the display.

4. The unit inflates the cuff until sufficient pressure has built up for a measurement. Then the unit slowly releases air from the cuff and carries out the measurement. When the device detects the signal, the heart symbol ⑯ on the display starts to flash, you can hear the beep for every heartbeat once the heartbeat signal is detected.

Please contact the distributor resp. the service centre if you can't solve the problem. Do not disassemble the unit by yourself.

5. When the measurement is finished, you can hear a long beep and the cuff is deflated completely. The systolic and diastolic blood pressure and the pulse value appear on the display ⑩. The blood pressure indicator ⑯ is displayed next to the relevant coloured bar depending on the WHO blood pressure classification. If the unit has detected an irregular heartbeat, the arrhythmia symbol ⑯ also appears.

WARNING
Do not take any therapeutic measures on the basis of a self measurement. Never alter the dose of a medicine prescribed by a doctor.

6. The readings are automatically saved in the previously selected memory (① or ②). Up to 120 measured values with date and time can be stored in each memory.

7. The measurement readings remain on the display. The unit switches off automatically after approx. 3 minutes if no buttons are pressed. The unit may also be switched off using the START/STOP button ④.

Transfer via Bluetooth® to VitaDock+® app

The MEDISANA Blood Pressure monitor BU 530 offers the possibility to transfer your measured values via Bluetooth® to the VitaDock+® app. The VitaDock+® app allows the evaluation, storage and synchronisation of your data between multiple iOS- and Android-devices to have access to it anywhere and anywhere. You may share your results with your friends or your doctor. Therefore you need a free user account, which you can create on the website www.vitadock.com. For Android or iOS mobile devices, you may download the respective apps. After each measurement an automatic transfer of the values will take place (provided that Bluetooth® is activated and configured on the receiving device).

During transmission, the symbol ⑯ " ⑯ " appears in the display. If the transmission has been successful, "OK" appears. If "Err" appears, a transmission error occurred. You may transfer the data also later on via Bluetooth® manually. Therefore, press and hold START/STOP button ④ when unit is switched off for approx. 5 seconds, until the symbol ⑯ " ⑯ " appears. The device will then automatically transmit all stored values for the actual selected user (① or ②).

Discontinuing a measurement

If it is necessary to interrupt a blood pressure measurement for any reason (eg. the patient feels unwell) the START/STOP button ④ can be pressed at any time. The device immediately decreases the cuff pressure automatically.

Display stored values

This unit features 2 separate memories, each with a capacity of 120 memory slots. Results are automatically stored in the memory selected by the user. Press the MEM button ③ when power off, to call up the measured values stored. The average values of the last 3 measurements appear on the display. Press the MEM button ③ again to call up the last measurement value. Repeatedly pressing the MEM button ③ displays the respective values measured previously. If you have reached the final entry and do not press a button, the unit automatically switches itself off after approximately 8 seconds in memory recall mode. Memory recall mode can be exited at any time by pressing the START/STOP button ④, which will also switch the unit off. The oldest value is deleted if 120 measured values have been stored in the memory and a new value is saved.

Memory - clear of measurements

If you are sure that you want to permanently remove all stored memories, press the SET button ⑤ (when power off) for approximately 3 seconds. Then press SET button ⑤ 6 more times until CL appears. Press the START/STOP button ④ and CL will flash for 3 times to clear all memory. Afterwards, press MEM button ③ and M and "no" will be shown on the display, what means that no memory is stored anymore.

Error messages and error remedying

Error indicators

The following symbol will appear on the display when measuring abnormal:

| Symbol | Cause | Correction |
|----------------|-----------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------|
| E-1 | Weak signal or sudden pressure change | Put on the cuff correctly. Repeat the measurement in the correct way. |
| E-2 | External strong disturbance | When near cell phone or other high radiant device, the measurement may fail. During measurement, do not move and do not speak. |
| E-3 | Error during inflation process | Put on the cuff correctly. Make sure, that the hose is properly connected. Repeat the measurement. |
| E-5 | Abnormal blood pressure | Repeat the measurement after relaxing 30 minutes. If you get unusual readings for 3 times, please contact your doctor. |
| Weak batteries | The batteries are too low or empty. Replace all 4 batteries with new 1.5 V type AAA LR03 batteries. | |

Cleaning and care

Remove the batteries before cleaning the unit. Clean the unit using a soft cloth lightly moistened with a mild soapy solution. Never use abrasive cleaning agents, alcohol, naphta, thinner or gasoline etc.. Never immerse the unit or any component in water. Be cautious not to get any moisture in the main unit. Do not wet the cuff or attempt to clean the cuff with water. Using a dry cloth, gently wipe away any excess moisture that may remain on the cuff. Lay the cuff flat in an unrolled position and allow the cuff to air dry.

Do not expose the unit to direct sunlight; protect it against dirt and moisture.

Do not expose the unit to extreme hot or cold temperatures. Keep the device in the storage case when not in use. Store the unit in a clean and dry location.

Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Remove the batteries before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your local authority or your supplier for information about disposal.

Guidelines / Standards

This blood pressure monitor meets the requirements of the EU standard for non-invasive blood pressure monitors. It is certified in accordance with EC Guidelines and carries the CE symbol (conformity symbol) "CE 0297". The blood pressure monitor corresponds to European standards EN 1060-1 and EN 1060-3.

The specifications of EU Guideline "93/42/EEC of the Council Directive dated 14 June 1993 concerning medical devices" are met.

Electromagnetic compatibility: (see attached leaflet)

Technical specifications

| | |
|---------------------------------------------|----------------------------------------|
| Name and model | MEDISANA Blood pressure monitor BU 530 |
| Display system | Digital display |
| Memory slots | 2 x 120 measurement values |
| Measuring method | Oscillometric |
| Power supply | 6 V; 4 x 1.5V batteries AAA LR03 |
| Blood pressure measuring range | 0 - 299 mmHg |
| Pulse measuring range | 40 - 199 beats/min. |
| Maximum error tolerance for static pressure | ± 3 mmHg |
| Maximum error tolerance for pulse rate | ± 5 % of the value |
| Pressure generation | Automatic with pump |
| Deflation | |

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